materialise

innovators you can count on

3-matic Medical 20.X

Instructions for Software Use – EN

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1 Instructions for software use - EN

Disclaimer: This document contains general instructions for use for 3-matic Medical. Additional details can be found in the user manual present in the application. English is the language used in the 3-matic Medical software interface.

Device description and performance characteristics

Materialise 3-matic Medical is a software that combines CAD tools with pre-processing (meshing) capabilities. To do this, it works on triangulated (STL) files which makes it suitable for freeform 3D data, like the anatomical data coming from the segmentation of medical images (from Mimics Medical). As such, 3-matic Medical can be seen as a tool that allows Anatomical CAD or what Materialise has coined 'Engineering on Anatomy'. 3-matic Medical may be used as a medical device, within the limits of the described below intended use statement.

3-matic Medical may be used to import anatomical data and other 3D objects in STL format or in numerous CAD formats including: CATIA, IGES, STEP, etc. Once objects have been loaded, they can be used for many applications of 'Engineering on Anatomy' including measuring, designing, modeling and 3D printing.

The modules that can be obtained for 3-matic Medical enable the users to do thorough 3D measurements and analyses, design an implant or surgical guide, or prepare the mesh for finite element modeling.

3-matic Medical can be extended with additional features via plugins. 3-matic Medical plugins may be used as a medical device, within the limits of the intended use of 3-matic medical.

3-matic Medical displays measurements with 4 digits behind the decimal point. Accuracy of measurements is limited by the data source.

US – Indications for use

3-matic Medical is intended for use as software for computer assisted design and manufacturing of medical exo- and endo-prostheses, patient specific medical and dental/orthodontic accessories and dental restorations.

EU & ROW – Intended purpose

3-matic Medical is intended for use as software for computer assisted design and manufacturing of medical exo- and endo-prostheses, patient specific medical and dental/orthodontic accessories and dental restorations.

Contraindications

No contraindications.

Intended user

The software should only be used by trained medical professionals.

Recommendations

To engage in the experience, we recommend that all users participate in a training offered by a Materialise expert.

Be advised that the quality and resolution of your triangle mesh can influence the result of design or analysis operations. For best results, we recommend that industry standards are referenced and followed.

Warnings

As stipulated in the End User License Agreement, Materialise warrants during the Warranty Period that 3-matic Medical shall operate substantially in accordance with the functional specifications in the Documentation. Any other warranty, whether express or implied, including but not limited to any warranty regarding fitness for purpose, safety, performance or effectiveness aspects of the medical application or device for which 3-matic Medical is used, is disclaimed.

Python scripts written by the user should always be validated by the user before being used for any purpose. Python scripts written for previous versions of 3-matic Medical should be re-validated by the user before being applied with the present version.

Precautions

R_x Only CAUTION: Federal law (USA) restricts this device to sale by or on the order of a trained medical professional.

Installation

Start the installation by launching the downloaded installation file (*.msi format). If this triggers the User Account Control to ask permission to make changes to your device, click 'Yes'. After this, the Setup Wizard will guide you through the installation. In every next dialog box, select your preferred options and choose 'Next' to continue. Do the actual installation by choosing 'Install', a progress bar will keep you up to date about the status.

If the application is installed successfully, the Finish dialog will appear, allowing you to complete the installation (click 'Finish').

If a new major, minor or bugfix release is distributed, installation is done in the same way as described above. The installer file will automatically overwrite (in case of a bugfix release) or install next to the already available versions. When starting the new version, you can choose whether or not to import the settings from the previous version.

Start

After successfully installing the software on your PC, double click the 3-matic Medical icon on your desktop. You may also click on the **Start Button** → **All Programs** → **Materialise** → **3-matic Medical** to begin. Be sure to select the *Medical edition* for clinical applications.

Materialise software is protected by an activation key. When you start 3-matic Medical for the first time or when your license has expired, the License Wizard will automatically start up to assist you with the registration process. Available options are explained in the **User Guide**.

Uninstallation

Uninstall can be done from the Windows Control Panel > Programs > Programs and Features menu.

Installing 3-matic Medical Plugins

First check if 3-matic Medical is installed and activated properly. From the Materialise website download the plugin and run the installer. After completing the installation wizard successfully, launch 3-matic Medical. You can verify correct installation by going to **Options & Help → About** and check if the plugin is listed in the about box. Before use, consult the user guide of the plugin available under **Plugins User Guides** menu.

Some plugins may require separate activation, so go to the **Licenses** wizard to initiate the activation process.

Minimum system requirements

Software	Hardware
Version 1909, Windows® 11 21H2	Ninth generation Intel® Core™ i7 or equivalent
Microsoft Edge® or equivalent	16 GB RAM
PDF viewer	DirectX® 12.0 compliant AMD Radeon/NVIDIA® GeForce® graphics card with 2 GB RAM
.NET framework 4.8 (or higher)	40 GB free hard disk space
Microsoft Edge® WebView2 (if Mimics Flow is used)	Resolution of 1920x1080 or higher

It is recommended to use our software within a hardware and/or network environment in which cyber security controls have been implemented including anti-virus and use of firewall. It is also recommended not to leave the workstation on which 3-matic Medical is running, unattended and to ensure the workstation is password-protected.

Note on additional, optional input devices: 3-matic Medical supports the use of the SpaceMouse by 3D Connexion. Please visit the manufacturer's website for the latest supported driver version for your specific model (<https://3dconnexion.com/nl/drivers/>).

Serious incident notice

For a patient or user in the European Union and in countries with identical regulatory regime (Regulation 2017/745/EU on Medical Devices): if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorised representative and to your national authority.

Cybersecurity incident notice

If a cybersecurity event is detected or suspected, please report it to the manufacturer through this email: mimics@materialise.com.

Additional information

Manufactured by:

Materialise NV
 Technologielaan 15
 3001 Leuven, Belgium
 Phone: +32 16 39 66 11

<http://www.materialise.com>

CH authorised representative:

MDSS CH GmbH
 Laurenzenvorstadt 61
 5000 Aarau
 Switzerland

Australian Sponsor:

AA-Med Pty Ltd; Suite 22.02, 101 Miller Street, North Sydney NSW 2060 Australia

India Importer:

Morulaa HealthTech Private Limited; No. 38, First Floor, Rajeswari Street, Santhosh Nagar;
 Kandanchavadi, Chennai 600096, Tamil Nadu, India

Registration number: Import License No. IMP/MD/2023/000231

Description of Symbols

	Legal Manufacturer
	CAUTION: Federal law (USA) restricts this device to sale by or on the order of a Physician
	Caution, please consult the instructions for use and the accompanying documentation
	Consult instructions for use or consult electronic instructions for use
	Date of manufacture
	Batch code
	Medical device
	Unique Device Identifier
	Catalog number
	Authorised representative in Switzerland
	CE marking

IFU for 3-matic Medical is supplied in electronic form in PDF format on www.materialise.com/electronic-instructions-for-use. Paper version may be requested by emailing leaflet@materialise.be and will be provided within 7 calendar days at no additional cost.

® Materialise, the Materialise logo and the Mimics and 3-matic product names are trademarks of Materialise NV.

© 2025 – Materialise N.V. All rights reserved.

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